



MARKING PLATES

With a marking plate mounted on the C-B60 or the Cool-B70 coil, the DLPFC spot is easy to locate during the motor threshold determination. When APB motor cortex is located, a curved line is drawn with a pen along the marking plate. The curved line from the marking has the shape of the treatment coil which makes it easy to position the treatment coil on the scalp over DLPFC.

Contact

Website: www.magventure.com

Email: info@magventure.com

Mechanical Data

Marking plate for Cool-B70 Coil	2.5 – 5 mm plastic. Based on 5.5 cm rule standard anterior to APB motor cortex
Marking plate for C-B60 Coil	1.5 mm plastic. Based on 5 cm rule standard anterior to APB motor cortex

Ordering Numbers

9016B064- (Cool-B70)

9016B019- (C-B60)